QUICK START GUIDE

BIG Candy Corn™ & LITTLE Candy Corn™ 2 accessibility switch

WARRANTY

AbleNet manufactured products include a 2-year limited warranty. This warranty is against defects in materials and manufacturing for 2-years from the date of purchase. Full warranty details available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(651) 294-3101
ablecare@ablenetinc.com
www.ablenetinc.com

Made with recycled content
EN July 26, 2021 8:52 AM

1 Product Registration

Registering your product provides you access to ableCARE, product updates, and resources for your product. Scan the QR code below to register your product.

2 Getting Started

Scan the QR code below to watch a short getting started video or follow the instructions listed.

To get started:

1. Connect switch plug to a device with a 3.5-mm / 1/8-in TS (mono) jack.
2. Turn switch on.
3. Switch will activate when a user is within 10-mm / 0.4-in of or touching the active proximity sensor area.

This device can do more! Full Instructions for Use are available at www.ablenetinc.com.

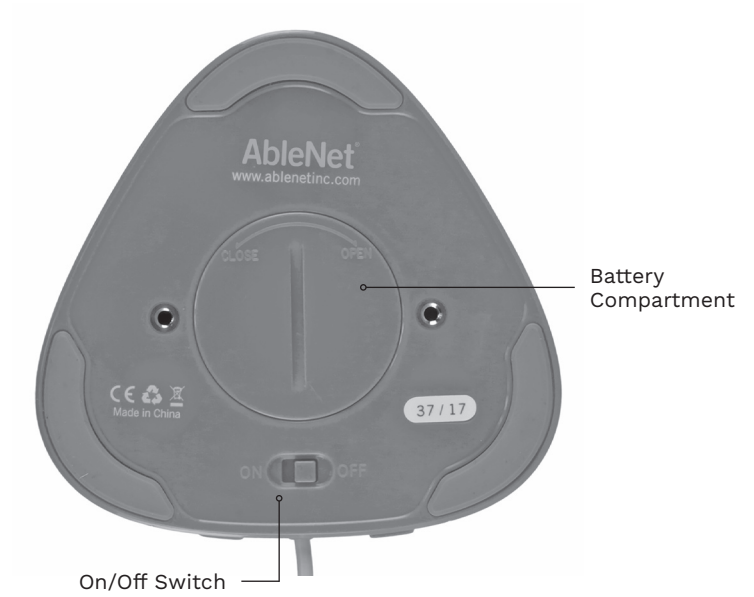
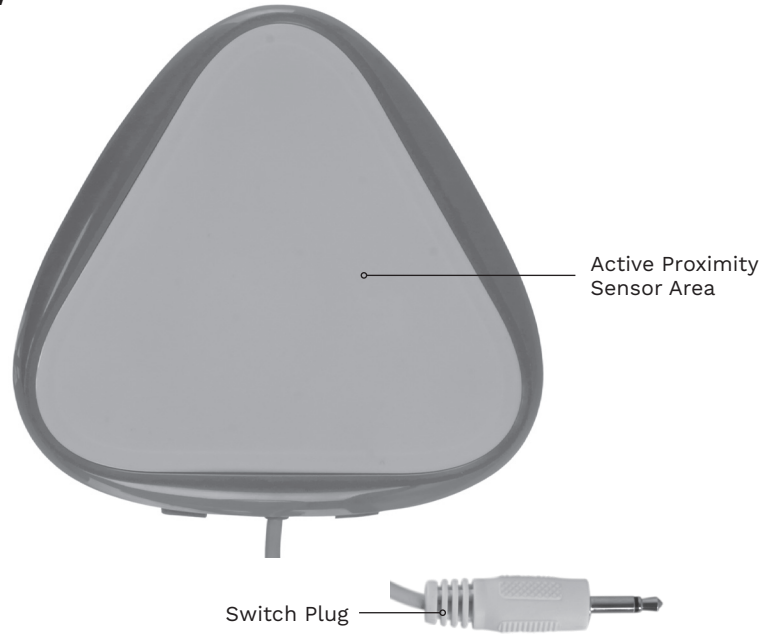
Download the ableCARE app to access live support from the ableCARE Product Success team, an online knowledge base filled with videos and getting started information, and other resources.

Scan the QR code below to download the free ableCARE app to your phone or tablet.

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