AbleNet[®]

Quick Start Guide

Hitch[®] 2


Computer Switch Interface

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
www.ablenetinc.com

*This device has been tested to comply with
FCC standards for home or office use.*

PN 98000053 (04/19 Rev. B - FN 70500247)

Product Overview

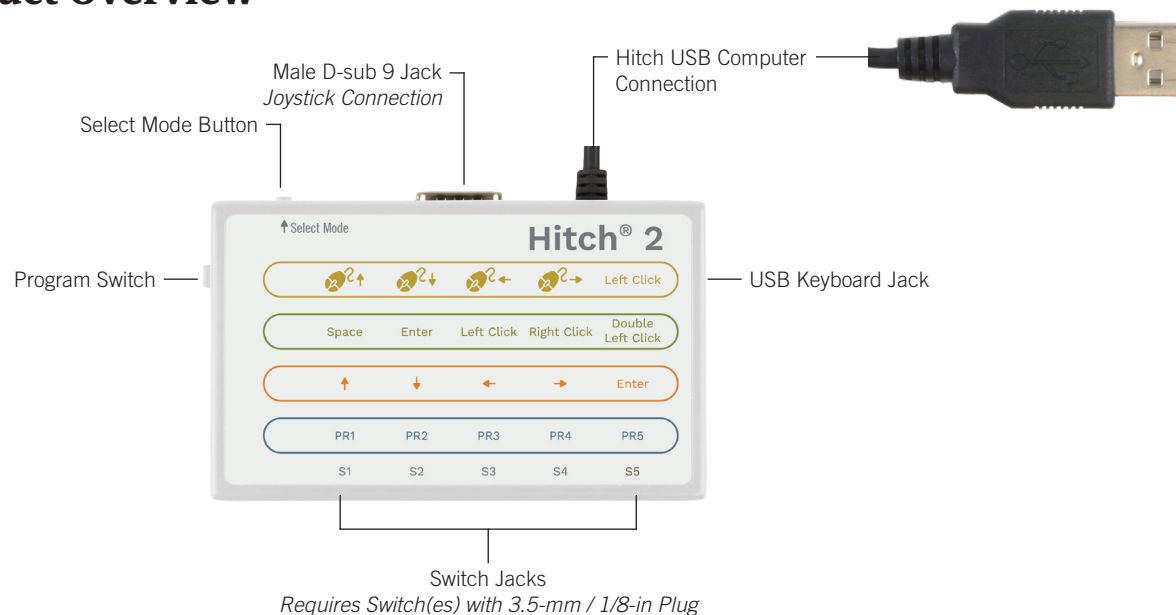Getting Started

Hitch 2.0 requires a computer with a USB jack and is compatible with most Windows, Mac OS X, Google Chrome, or Android operating systems.

1. Connect Hitch 2.0 to USB port on computer. One LED light on Hitch 2.0 will turn on when connected.
2. Activate Select Mode button to select desired row of functions
3. Attach switch(es) with 3.5-mm / 1/8-in plug or joystick with female D-sub 9 connector to Hitch 2.0
4. Activate switch(es) or joystick to send function to computer

Custom Program (Optional)

The functions PR1 to PR5 found in the fourth / blue mode on Hitch 2.0 can be programmed to send one to four keyboard keystrokes of your choice to a computer. Complete the following steps to program Hitch 2.0.

1. Connect Hitch 2.0 to USB port on computer. One LED light on Hitch 2.0 will turn on when connected.
2. Activate the Select Mode button and select the fourth / blue mode with the functions PR1 to PR5
3. Attach switch(es) with 3.5-mm / 1/8-in plugs into Hitch 2.0
4. Move the Program switch on Hitch 2.0 to On
5. Plug a USB keyboard into Hitch 2.0
 - a. Note: If using a Mac OS X computer, please make sure you first plug the USB keyboard directly in the Mac OS X computer and go through the Keyboard Setup Assistant so that the computer can identify the keyboard. Once the Keyboard Setup Assistant is complete, unplug the USB keyboard from the Mac OS X computer and plug it directly into Hitch 2.0. Failure to do so may result in the keyboard not working with Hitch 2.0.
6. Select the switch plugged into Hitch 2.0 that you would like to program and hold it down until the blue LED light begins to blink on and off
 7. While the blue LED light is blinking on and off, select one to four keystrokes on the USB keyboard
 - a. If you select one, two, or three keystrokes, activate the same switch a second time to exit recording mode
 - b. If you select four keystrokes, Hitch 2.0 will automatically exit recording mode
 8. Repeat steps 6 and 7 if you would like to program additional switches connected to Hitch 2.0.
 9. After the programming of each switch is complete, move the Program switch on Hitch 2.0 to Off and disconnect the USB keyboard from Hitch 2.0
 10. Hitch 2.0 is now ready to use your custom keystrokes with each switch

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc. 2625 Patton Road, Roseville, MN 55113
(800) 322-0956 / www.ablenetinc.com

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FCC

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

FCC logo

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Consult AbleNet Technical Support or your local Distributor for help.

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FCC logo

For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that

used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product, to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment(EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union(EU). If you wish to discard this product please contact your local authorities or dealer and ask for the correct method of disposal.

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