DE/EN

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**INSTRUCTIONS
FOR USE**

AssistX IR one

C-SS
MICRO SYSTEMS

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WEEE DE94635635

Table of Contents

1. Introduction	21
2. Intended use	21
3. Explanation of symbols	22
4. Safety instructions	23
5. Cleaning and disinfection	26
6. Scope of delivery	27
7. Device Buttons, Ports, Display	28
8. Use of the device	30
9. Settings	30
Address settings:	30
10. Maintenance / Review	32
11. Support	32
12. Technical specifications	33
13. Conformity / Standards	33
14. Environmental protection & disposal	34

1. Introduction

Thank you for choosing a high quality CSS product. Please read these instructions for use carefully and take care of these instructions when using the device. Please keep the instructions for use at hand when you use the device.

2. Intended use

AssistX IR one is a sensor that responds to infrared signals. When a specific IR code (4 addresses can be set) is received, a switching function is sent to the **AssistX Port**, whereupon **AssistX Call** or **AssistX Mobil** triggers a call. Therefore, **AssistX IR one** is an ideal extension for infrared adaptive EC or AAC devices. **AssistX IR one** is self-monitoring and triggers via the **AssistX Port** in the event of a failure. **AssistX IR one** is magnetic at the bottom of the housing for easy mounting on pipes or optional CSS mounting components.

- **AssistX IR one** reacts to a programmed infrared signal at 38 kHz
-

- The signal is transmitted with one of four different IR addresses (so up to four **AssistX IR one** devices may be used in one room)
- **AssistX IR one** provides a send function to teach environmental control devices the used IR signal
- **AssistX IR one** is self-monitored, in case of a malfunction the device triggers a call

3. Explanation of symbols

Safety instructions!

Important instructions!

4. Safety instructions

Handling

Please handle the device carefully. The device contains sensitive electronic components. It is not allowed to do any modification. The device is not life-sustaining or life-supporting. The device consists of several components. Small items may be detached from the device, which may result in suffocation or other danger to children. Small children or people with intellectual disabilities should not reach or use the device without supervision.

Do not use the device in damp, wet, very cold or very hot environments (see temperature information in technical data).

Never replace the built-in batteries by yourself! The battery can be damaged, which can lead to overheating and can lead to injuries. The integrated rechargeable lithium polymer battery may only be replaced by authorized personal.

Mounting

The device must be mounted in a way that it cannot fall down. Only recommended mounting components from the manufacturer are to be used. The user bears the sole risk of mounting.

The manufacturer is not liable for personal injury or property damage caused by the device or its components falling down.

Repair

Under no circumstances you should open the device and repair it by yourself. Disassembling the device can permanently damage the device and cause injury. Contact your reseller or the manufacturer if the device is damaged or does not work properly. If the device or its accessories are mechanically damaged, they may no longer be used.

Interaction with medical devices

The device and its components can emit electromagnetic radiation and have integrated magnets. The device must therefore not be placed on a person or brought closer than 15 cm to people using other active medical devices. The device must also not be placed directly on active medical devices. Please pay special attention to people with implanted pacemakers. If necessary, please contact the device manufacturer of the other active medical device.

Attention! Magnets

The mounting plate contains a strong neodym magnet. Magnets can affect cardiac pacemakers and implanted defibrillators.

- The pacemaker could be switched into testmode and cause discomfort
- A defibrillator could stop working
- Keep enough distance to magnets if you use such devices
- Advice other users of such device against getting too close to magnets
- Keep magnets away from all devices and objects, which can be damaged by strong magnetic fields

Recommended safety distance is 6 inch (15 cm).

Magnets cause a far-reaching magnetic field. They could damage TV sets, laptops, PC harddrives, creditcards, ATM cards, data storage, mechanical watches, hearing aids and speakers.

Vehicle keys, USB sticks, cameras, cell phones and smart phones contain non-magnetic storage media. Therefore, static magnetic fields near those devices cannot delete data. There are countless covers with magnetic closures or magnetic holders on the market, which also speaks against the notion that magnets can damage these devices. However, it cannot be ruled out that very strong magnetic fields might magnetise and maybe damage mechanical parts or built-in speakers. When in doubt, keep these devices away from strong magnets.

5. Cleaning and disinfection

Before cleaning the device, switch it off and disconnect all cables. The device should be cleaned if required with a disinfectant and cleanser intended for use in hospitals. Cleaning should only be done with a wipe. Wipe all surfaces and let them dry or follow the wiping instructions.

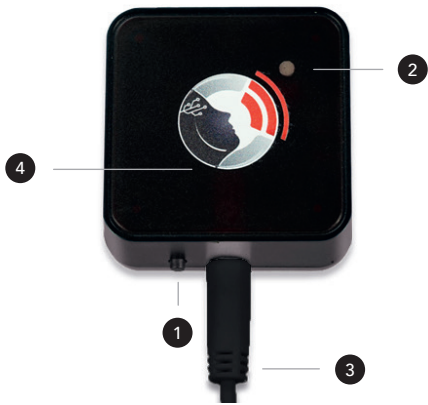
Do not use sprays that may allow moisture to enter the device. Under no circumstances may any component of the device be subjected to steam or wet sterilization.

6. Scope of delivery

- **AssistX IR one**
- 2 m 4pol. connection cable
- Instructions for Use

7. Device Buttons, Ports, Display

-
- 1 Button
 - 2 LED
 - 3 Connection to **AssistX Port**
 - 4 Position of the infrared transmitter
-

① **LED Display** ● settings ●●●● change ●●●● confirm

① LED Display: Addresses

●	Addr 1	●	Addr 1
● ●	Addr 2	● ●	Addr 2
● ● ●	Addr 3	● ● ●	Addr 3
● ● ● ●	Addr 4	● ● ● ●	Addr 4

8. Use of the device

Connect the device with the enclosed appropriate cable to the **AssistX Port** of your CSS product and follow the steps of 9. Settings. Available AssistX stations are **AssistX Call**, **AssistX Mobil** or Click-Man (the newest version with red sensor receiver).

9. Settings

AssistX IR one reacts to infrared signals with a predefined address and signalises the AssistX station when the predefined IR signal is received. You only have to set the addresses.

Address settings:

When a specific IR code (4 addresses can be set) is received, a switching function is sent to the **AssistX Port**. Each address can also be sent to teach in an environmental control device.

- **set address**
Press the ① Button 1 sec Release the button when the ② LED starts blinking.
 - **change address**
Press the ① Button shortly. With each release the ① address is changed. The ② LED shows the selected address 1-4 times per sec in **yellow**.
 - **confirm address**
If the right ① address is set, press the ① button for 1 sec The ② LED shows the selected address 1-4 times per sec in **green**.
 - **teaching infrared-code**
The IR code is sent to the environmental control device until the ① button is released.
-

10. Maintenance / Review

The device usually does not require any maintenance. However, we recommend the following **every 3 months**:

- functional test
- Connectors for damage and soiling
- Damage to the housing

11. Support

If you need any support or for technical questions about the product, please contact the reseller where you purchased this product. If necessary, e. g. the reseller is not available or cannot help, send us an email to support@csslabs.de. Please be sure to include a description of the problem and the serial number of the device.

12. Technical specifications

Dimensions: 50 x 50 x 26 mm

Weight: 60 g

IP Rating: IP 54

Power supply: 12 VDC via **AssistX Port**

Connection: **AssistX Port**, 3.5 mm jack, 4 pin cable

Valid IR signals: at 38 kHz

Temperature ranges

Use: 0 °C to 45 °C

Transportation and storage: -10 °C to 50 °C

13. Conformity / Standards

 This product is CE compliant.

AssistX IR one meets the requirements of DIN VDE 0834 in conjunction with **AssistX Call** and **AssistX Mobil**.

14. Environmental protection & disposal

All electrical and electronic devices must not be disposed with general household waste, but must be disposed at designated disposal points. This product is subject to the European Directive 2002/96/EC and must therefore be properly disposed at the WEEE collection points.

WEEE DE94635635

THE PASSIONATE ASSISTIVE TECHNOLOGY MANUFACTURER

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© 2021 CSS MicroSystems GmbH