Big Jack

User manual

Art. Nr: 402500

Index

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<i>Återställning till grundinställning</i>	17
GB: English Manual Gewa Big Jack IR-6SP	19
Introduction	19
Intended purpose	19
General Information about Big Jack.....	19
Overview of Big Jack.....	20
Description of Symbols	20
Inserting the Batteries	21
Programming	22
Recording IR Channels from another Transmitter	22
<i>Recording IR channels</i>	23
Setting beep volume	24
<i>Setting beep volume</i>	24
Setting an Acceptance Delay	24
<i>Setting an acceptance delay</i>	24
Creating an automatic sequence (Macro).....	25
<i>Creating an automatic sequence (Macro)</i>	25
Resetting to Factory Setting.....	26
<i>Resetting to default setting</i>	26
External Control Switches.....	27
Cleaning.....	27
Accessories.....	27
Accessories included with Big Jack	27
Other accessories for Big Jack	27
Technical data.....	27
Important information	28
Serious incidents.....	28
Conformance with requirements for medical devices	28
Product liability	28
Storage and disposal	28
Storage.....	28
Recycling Waste Electrical and Electronic Equipment.....	28
Summary Quick Guide Gewa Big Jack.....	29
<i>Recording IR channels</i>	29
<i>Setting beep volume</i>	29
<i>Setting an acceptance delay</i>	29
<i>Creating an automatic sequence (Macro)</i>	29

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GB: English Manual Gewa Big Jack IR-6SP

Introduction

As the name implies Big Jack is a large and robust remote control device to which external control switches can be connected. It can be programmed, which means that it is possible to "teach" it IR codes from other remote controls, such as those used for TVs, videos and stereo systems. It is suitable for use by people who have cognitive and motor difficulties, and who require large and clear control buttons.

Intended purpose

Big Jack is a robust remote control intended to support people who are unable to use a standard remote control due to motor or cognitive impairments. With up to six connected external control switches, the device can help the user control electronic equipment at home, including media devices and toys. The device is intended for indoor use.

General Information about Big Jack

There are certain things you should know before starting to use Big Jack.

To enter the programming mode, press the Programming button using a paper clip, for example, **at the same time** as you choose a *Function*-button. Different *Function*-buttons are chosen depending on what is to be programmed.

If you press the wrong button or make a mistake, when programming, you can always press the Programming button for two seconds. It works as a *Reset*-button.

It is important not to wait too long between the different steps when programming. After a certain amount of time Big Jack automatically leaves the programming state to save power.

Overview of Big Jack

Fig 1


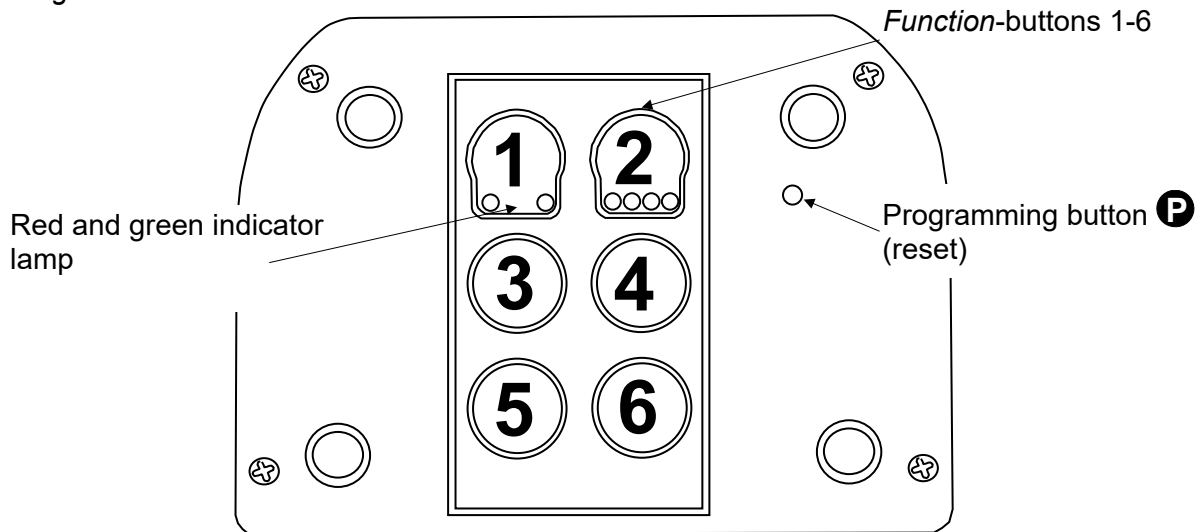
Fig 2

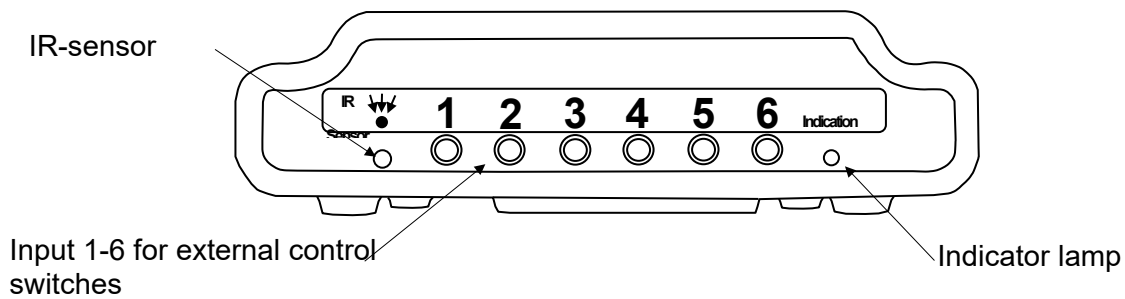Certain basic settings have the following factory defaults when Big Jack is delivered:

The beep volume is set to 3.

The acceptance delay is set to 1, meaning no delay.

GewaLink channels 16-21 are programmed to the buttons 1-6.

Description of Symbols

The following symbols appear in the text throughout this manual:

Programming button

Function button 1-6

Any button on your Big Jack

Any button on another transmitter to be programmed

Inserting the Batteries

Unscrew the screws as shown in *Fig 3* using a Philips screwdriver. Remove the upper part and insert four 1.5 V alkaline batteries of type LR6. Insert the batteries with the correct polarity as shown on the bottom of the battery compartment. It is important to use **alkaline** batteries that are protected from leakage and that have a long lifetime.

Finish inserting the new batteries by **pressing P**.

Fig 3


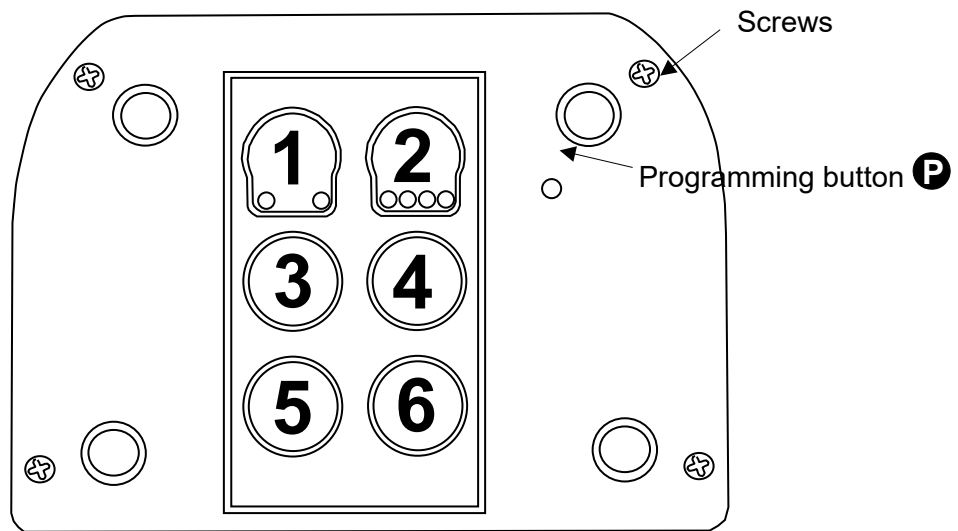
Fig 4

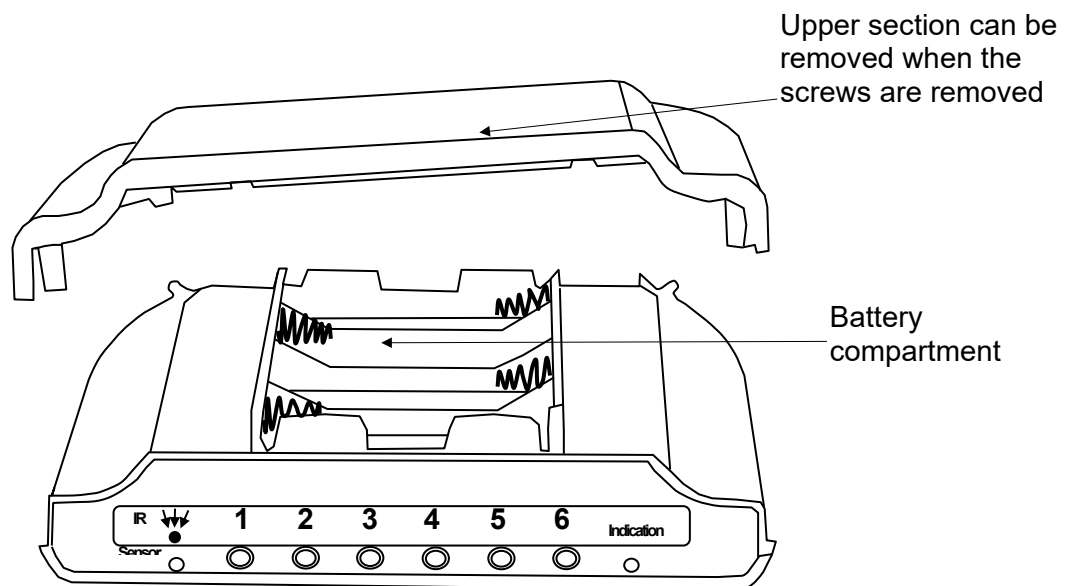Programming

Recording IR Channels from another Transmitter

In order for Big Jack to control a TV set, video, stereo unit, DVD, toys, lamps, etc., it is necessary that these appliances have a remote control that emits IR. Most remote control units on the market use infrared light (IR), as does Big Jack. A TV set, for example, always has a remote control as standard, which is used to control various functions. It is this remote control unit that is to be used when teaching Big Jack the IR channels. Other appliances such as lamps, hall telephones, telephones and automatic door openers do not have such a remote control unit. Abilia supplies solutions that make it possible to control such appliances. Contact Abilia or a representative of Gewa in such cases.


Proceed as follows to teach Big Jack IR codes from another remote control unit:

Check that the batteries in Big Jack and in the remote control unit are fully charged. Shield the units from intense light such as direct sunlight during the teaching procedure.

Place Big Jack on a table with the remote control unit pointing at the IR sensor on Big Jack, as shown in *Fig 5*.

Press **P** and **1** at the same time.

In order to check that Big Jack and the remote control unit are positioned correctly, press and hold down any button on the remote control unit. Move the remote control unit left and right until the green indicator lamp goes out. Then move the remote control unit backwards until the indicator lamp goes out (1 metre for certain units). Place the remote control unit between these limits. Certain types of remote control unit give only a short flash for each press. If this is the case, press the button on the remote control unit repeatedly.

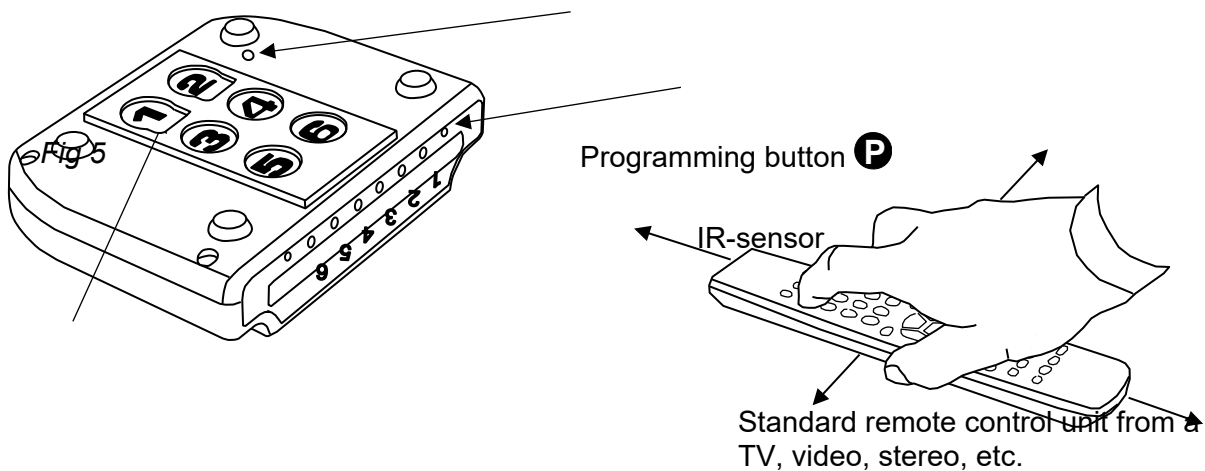Green and red indicator lamp

Press and release the button **X** on Big Jack on which the IR channel is to be placed. The red and green indicator lamp now lights with a steady light.

Press and hold down the button **Y** on the remote control unit that is to be taught to Big Jack. Release it when the red indicator lamp goes out. This may take as long as 10 seconds. If the red indicator lamp returns to a red light, press again one or several times until both red and green indicator lamps go out.

If the indicator lamp flashes red and a sound signal is heard ♪♪♪♪ , the teaching process has failed. Repeat the process from step 4.

Conclude the process by pressing button **6**.

If the teaching process has succeeded for the first button, continue with the next button as described above in points 3-6 until all IR channels are programmed.

TIPS: Big Jack can learn IR channels from most remote control units on the market, but there are a few exceptions. It may affect certain functions or it may affect all of the functions on a remote control unit. It is possible in these cases to use another teaching mode. Replace step 3 above by the following procedure: press in **P** and **1** at the same time and hold them depressed for about 2 seconds until the green indicator lamp flashes and then goes out. The unit is now in a different learning mode. Continue with the procedure described above.

Recording IR channels

P + **1**, select button on Big Jack **X**, select button on standard remote control **Y**, then continue or finish with **6**.

Setting beep volume

Big Jack has a beep signal which sounds when a button is pressed. Set the beep volume in 6 stages (1, 2, 3, 4, 5, 6).

1 = no sound at all and 6 = maximum sound level.

Do as follows:

Press **P** and 4 at the same time.

Press a button from 1 - 6 to choose beep volume.

The indicator lamp for button 1 flashes green to show that the operation is complete.

Setting beep volume

Press **P** + 4, choose the setting.

1 = no sound at all and 6 = maximum sound level.

Default setting: Beep Volume = 3 (1 – 6)

Setting an Acceptance Delay

Big Jack makes it possible to set an acceptance delay. By acceptance delay is meant the time a button has to be pressed before a result is reached. The acceptance delay can be set in 6 steps (1, 2, 3, 4, 5, 6).

1 = no delay time and 6 = maximum delay time (0.5 second).

Do as follows:

Press **P** and 2 at the same time.

Press a button from 1 - 6 to choose delay time.

The indication lamp for button 1 flashes green to show that the operation is complete.

Setting an acceptance delay

Press **P** + 2, choose setting.

1 = no delay time and 6 = maximum delay time (0.5 second).

Default setting: Acceptance delay = OFF (0 sec).

Creating an automatic sequence (Macro)

Your Big Jack is able to send sequences of codes. When you press **one** button, Big Jack automatically carries out what is equivalent to pressing several buttons. One available application for instance is muting the TV at the same time as you answer the telephone. Your Control 18 can also be programmed to change levels and it remembers how long a button should transmit.

The transmission of a sequence can be terminated at any time by pressing another optional button.

When a sequence is programmed your Big Jack transmits at the same time what is already on the button. Max. 25 seconds and 6 button presses per sequence can be programmed.

NOTE: Buttons 5 and 6 cannot store sequences, and the button on which a sequence is stored cannot be included in the sequence.

Press **P** and **5** at the same time.

Press **1**.

Press the buttons which should be used for the sequence. Press as long as the function should be transmitted when the sequence is played.

Press **P** and **5** at the same time.

Press **2**.

Choose the button **X** where you want the sequence to be stored (not button 5 or 6).

The indication lamp for button 1 flashes green to indicate that the operation is complete.

Creating an automatic sequence (Macro)

Press **P + 5**, **1**, create the sequence, **P + 5**, **2**, choose button **X**.

Max 25 seconds per press, 6 presses

Resetting to Factory Setting

It is possible to reset all of Big Jack's buttons and settings to the factory defaults, shown below:

Beep Volume reset to value 3.

Acceptance delay reset to value 1, no delay.

GewaLink channels 16-21 programmed to the *Function* buttons 1-6.

Proceed as follows to reset to factory defaults:

Press **P** and **3** at the same time.

Press in turn the buttons **5**, **1**, **2**, **3**, **6**.

The indication lamps for button 1 flashes green to show that the operation is complete.

Resetting to default setting

Press **P** + **3**, **5**, **1**, **2**, **3**, **6**.

External Control Switches

It is possible to connect external control switches to Big Jack, which influence buttons 1-6. The control contacts are connected to the inputs on the front using 3.5 mm telephone plugs.

Cleaning

Big Jack may be cleaned with a rag that has been immersed in water with a small amount of washing up liquid and then **wrung dry extremely thoroughly**. Big Jack may be dried with a dry rag after cleaning.

Accessories

Accessories included with Big Jack

Art. no.	Name	Description	Quantity
-----	Note sheet Big Jack	A4 sheet for recording programmed channels	1

Other accessories for Big Jack

Art. no.	Name	Description
8362	Battery LR6	1.5 V Alkaline

Technical data

Power consumption at rest:	Approx. 10 μ A
Power consumption at transmitting	Approx. 35 mA (GewaLink channel)
Battery life:	Approx. 2 years (200 seconds transmitting per day with GewaLink channel)
Programming locations:	6
Operating force buttons:	ca 1,5 N (150 gr)
Range:	20-30 m (GewaLink channels)
Measurements (L x W x H):	140 x 100 x 38 mm
Weight (incl. batteries):	280 gram
Operating temperature (excl. batteries):	-20°- +45°C
Sockets for connecting control switches:	3.5 mm

Important information

Before you use the product read and understand the contents of this user manual.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Conformance with requirements for medical devices

Big Jack is marketed as a technical aid for people with disabilities. It complies with all necessary requirements, regulations and directives for medical devices.

Product liability

The product is developed and risk-assessed according to ISO 14971. The manual and the technical specifications are made according to the safety assessments in the risk analysis. Always consider the safety when using the product. Abilia AB does not take responsibility for any consequences following incorrect use or installation of the product. Any form of damage or tampering with the product invalidates the warranty and disclaims the manufacturer's liability. The estimated lifetime of the product applies when used in accordance with its intended use and maintenance instructions.

Storage and disposal

Storage

Keep the product indoors at room temperature and away from moist.

Recycling Waste Electrical and Electronic Equipment

The IR-6SP Big Jack contains electronics components that must be recycled. When the product is no longer of use, hand it in to a recycling centre for waste electrical and electronic equipment.

Summary Quick Guide Gewa Big Jack

Recording IR channels

Press **P** + **1**, select button on Big Jack **X**, select button on standard remote control **Y**, then continue or finish with **6**.

Setting beep volume

Press **P** + **4**, choose the setting.

1 = no sound at all and **6** = maximum sound level.

Default setting: Beep Volume = 3 (1 – 6)

Setting an acceptance delay

Press **P** + **2**, choose setting.

1 = no delay time and **6** = maximum delay time (0.5 second).

Default setting: Acceptance delay = OFF (0 sec).

Creating an automatic sequence (Macro)

Press **P** + **5**, **1**, create the sequence, **P** + **5**, **2**, choose button **X**.

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Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Abilia UK Ltd, Unit 12 Buckingham Business Park, Swavesey,
Abilia AB, Råsundavägen 6, Solna, Sweden